

Miltex

REF 5-5000

The Miltex Edge™

Scissors Sharpener

LOT



+H834550000/

Contains 1 Sharpener

UM EA

Manufacturer
Miltex, Inc.
York, PA 17402

EU Representative
Miltex GmbH
78604 Rietheim-Weilheim, Germany

MADE IN USA

cut line

Miltex

REF 5-5010

The Miltex Nip-It™

Nipper and Bone Cutting
Forceps Sharpener

LOT



+H834550100+

Contains 1 Sharpener

UM EA

Manufacturer
Miltex, Inc.
York, PA 17402

EU Representative
Miltex GmbH
78604 Rietheim-Weilheim, Germany

MADE IN USA

cut line

fold line

The Miltex Edge™ REF 5-5000

Directions:

1. Grasp the Miltex Edge™ in left hand with curved edge resting firmly against palm. (for better control, we recommend the Miltex Edge™ to be held firmly on a flat surface, i.e. a table) *Figure 1*
2. Grasp opened scissors in right hand and carefully hold open by holding at the screw-lock.
3. Place flat side of inside blade against sharpener with cutting edge resting on sharpening stone. *Figure 2*
4. Run blade through sharpener. Start near screw and pull all the way through in one motion. Repeat motion 2 or 3 times until blade is sharpened. A very dull blade may require additional sharpening motions.
5. Turn scissors to other blade and repeat.

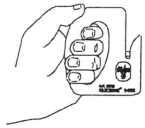


Figure 1

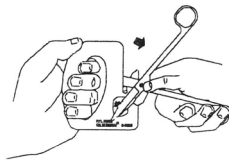


Figure 2

* Not intended for SuperCut or serrated edge scissors

NOTE: Proper use of Miltex Edge™ does not impact instrument warranty

L5-5000UEU

REV. A 03/08

The Miltex Nip-It™ REF 5-5010

Directions:

1. Grasp the Miltex Nip-It™ with curved edge resting firmly against palm. (for better control, we recommend the Miltex Nip-It™ is held firmly on a flat surface, i.e. a table)
2. Turn nipper or bone cutting forceps over so that the flat surface of jaw (back of jaw) is facing up. *Figure 1*
3. Place opened instrument at top (small end) of sharpening stone.
4. Draw opened instrument from top to bottom of stone, pulling instrument toward you with a circular downward motion. Repeat 2 or 3 times until blades are sharpened. *Figure 2*
5. Pass back (flat side) of closed instrument over stone to remove any burs.

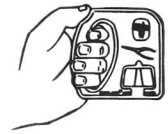


Figure 1

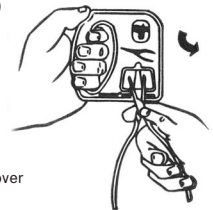


Figure 2

NOTE: Proper use of Miltex Nip-It™ does not impact instrument warranty

L5-5010UEU

REV. A 03/08