

**PRODUCT USE:**

The MILTEX BIOPBLADE® is utilized for removal of lesions, either elevated (shave biopsy) or flat (saucerization).

**INDICATIONS FOR USE:**

The MILTEX BIOPBLADE® is indicated for use by a licensed medical professional to obtain a biopsy of any soft tissue. This is a disposable product and is not sterilizable.

**WARNINGS/PRECAUTIONS:**

- Use caution when handling and disposing of this product as the blade is sharp.
- Do not use this product if the individual bag is broken as sterilization cannot be guaranteed.
- The MILTEX BIOPBLADE® is a disposable product intended for single use applications.
- Do not place on or near any products containing chlorides as rusting may occur.

**CONTRAINDICATIONS/LIMITATIONS:** None known.

**ADVERSE REACTIONS OR SIDE EFFECTS:** None known.

**HOW SUPPLIED:** Each MILTEX BIOPBLADE® is individually sealed in a pouch to ensure sterility. These products are sold in boxes of 50.

**STEP-BY-STEP INSTRUCTIONS:**

1. After the site is anesthetized, remove MILTEX BIOPBLADE® from its packaging.
2. Hold the MILTEX BIOPBLADE® “bowed” between the thumb and fingers. (see fig. 1)
3. NOTE the “UP” on the MILTEX BIOPBLADE®. This should face up. (see fig. 2)
5. Remove the lesion at or just below the surface epithelium.
6. Dispose of the MILTEX BIOPBLADE® as you would dispose of a used needle.

**NOT RECOMMENDED/INAPPROPRIATE USES:** None known.

**OTHER IMPORTANT INFORMATION:** None known.

**SYMBOLS USED ON LABELING**

See Instructions for Use

Consult Instructions for Use

Use by: Year-Month

Single Patient Use

Batch Code / Lot Number

Catalog Number

To be used by or on the order of a licensed medical professional.

Authorized Representative in the European Community

STERILE – STERILIZED BY IRRADIATION  
Sterility guaranteed unless package opened or damaged. Do not re-sterilize.

Product complies with requirements of directive 93/42/EEC for medical devices

Manufactured for:  
Miltex, Inc.  
589 Davies Drive  
York, PA 17402  
toll free phone: 866-854-8300  
phone: 717-840-9335  
fax: 717-840-9347  
www.miltex.com

**Miltex GmbH**  
Mühlstr. 6  
78604 Rietheim-Weilheim,  
Germany  
Phone: 49 7461 /9634-14  
Fax: 49 7461 /9634-30

1209/0/B/2

SURG-BIOPBL-DFU



\*+H834SURGBIOPBLDFU0+\*

fig. 1



fig. 2

