# Integra™ Miltex® Instrument Cassette Directions For Use

**Rx ONLY**

## Directions For Use

Integra™ Miltex® Instrument Cassettes

## Device Description

Integra™ Miltex® Cassettes are sterilization cassettes made of surgical-grade passivated stainless steel to provide protection, storage and organization for all types of instruments. Integra™ Miltex® Cassettes may be used in dental, surgical or veterinary procedures. Cassettes are available in a variety of configurations and are designed to fit standard autoclaves.

## Indications For Use

Integra™ Miltex® Cassettes are general dental/surgical instrument cassettes indicated to hold instruments and accessories in place during storage and the sterilization cycle. Suitable for gravity steam and pre-vacuum steam, the cassettes are intended to be used in conjunction with sterilization wrap in order to maintain sterility.

## Contraindications

- None

## Warnings

- Do not stack or overfill instruments in cassette; ensure that each instrument is correctly placed into slots provided.
- Close cassette lid to avoid injury caused by instruments protruding from cassette.
- Do not stack cassettes in sterilization unit.
- Do not use any unused instruments in the cassette once soiled instruments have been placed back into the cassette.

## Precautions

- Wrap instrument cassette only with legally marketed FDA-cleared sterilization wrap.
- Use of nonabsorbent tray liners, such as plastic/silicone-fingered organizing mats, can cause condensate to pool inside cassette.
- Small baskets, trays, and other types of accessories, especially those with covers or lids, should be used with cassettes only if cassettes have been specifically designed and tested for such purpose.
- Complex instruments should be prepared and sterilized according to instrument manufacturer's instructions.
- For effective sterilization and drying, the weight of the wrapped surgical cassette and its contents should not exceed 4.4 lbs per the validated sterilization study.

## Directions For Use

### Cassette Cleaning

1. Cassettes should be cleaned with an instrument brush and surfactant or detergent and water to remove visible debris from all crevices prior to use.
2. Cassettes may be cleaned in an ultrasonic cleaner with an enzymatic solution, such as Integra™ Miltex® EZ-Zyme All Purpose Enzyme Cleaner.
3. Submerge cassette fully into ultrasonic unit and place lid on unit.
4. Process single layer cassettes for 15 minutes and double layer cassettes for 30 minutes in ultrasonic unit.
5. Following ultrasonic cleaning procedure, remove cassettes and thoroughly rinse with warm water to remove cleaning solution; allow cassette to dry before placing instruments in cassettes for processing.

### Manual Cleaning - Instruments

1. Open instrument cassette by pressing push button to release lid. Fold lid under cassette or lay lid flat on a table or work station.
2. Remove visible debris from each reusable instrument by scrubbing instrument manually with an instrument brush and surfactant or detergent and water.
3. Rinse with water to remove chemical or detergent residue.
4. Dry instruments and place them into cassette. Hand instruments fit into grooves on racks inside cassette; additional instruments can be placed in accessory area of cassette.
5. Close cassette lid tightly after placing instruments in cassette.

## Sterilization

1. Wrap cassette in FDA-cleared sterilization wrap; seal wrap with autoclave tape.
2. Date and code cassette on autoclave tape or label on outside of wrapped cassette.
3. Place wrapped cassette into sterilization unit and follow sterilizer's recommended sterilization procedures and an approved National/International Standard.

Integra™ Miltex® cassettes have been tested with no adverse effects under the following sterilization parameters:

### Gravity Steam:

Temperature: 250°F (121°C)  
Exposure Time: 30 minutes  
Dry Time: 20 minutes

### Pre-Vacuum Steam:

Temperature: 270° F (132°C)  
Exposure Time: 4 minutes  
Dry Time: 20 minutes

4. After sterilization, remove cassette from sterilization unit and place in a storage location. Cassettes wrapped in FDA-cleared sterilization wrap maintain sterility of enclosed instruments for 30 days as long as sterile barrier is not compromised.

## Maintenance Of Cassette

- Integra™ Miltex® cassettes have been validated to withstand up to 50 sterilization cycles with no evidence of deterioration.
- For best and lengthy service life, instrument cassettes should be allowed to properly dry before the sterilization procedure.
- Use Integra™ Miltex® Instrument Stain Remover to remove water marks or stains if they appear on cassette.

## How Supplied

Integra™ Miltex® Cassettes are supplied one per package in a variety of configurations.

## Symbols Used On Labeling

Manufacturer

Catalog number

Lot number

CAUTION: See warnings or precautions

Consult instructions for use

**Rx ONLY**

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a dentist or physician.

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Integra York PA, Inc. ■ 589 Davies Drive, York, PA 17402