Rx ONLY

Rx ONLY

Rx ONLY


[REF] 75-20 Screw type [REF] 75-22 Snap on

Precautions

- The contra angle must be lubricated before every autoclaving procedure.
- Damage due to lack of lubrication is misuse and is not covered by the warranty.
- Maximum autoclave temperature is 270°F.
- Do not use dry heat or cold sterilization.

Lubrication Instructions

1. Remove cup or brush.
2. Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
3. Unscrew the knurled head cap by turning clockwise and remove the cap and screw stem holders. The snap on style angles have a single unit cap, ball and gear assembly.
4. Lubricate all gears with a hand piece lubricant and reassemble, making sure that the knurled cap is tightened firmly counterclockwise.
5. Rotate the shaft several times to circulate the lubricant and to make certain that the gears have intermeshed properly.
6. Wipe off any excess lubricant expelled.
7. Sterilize by autoclaving before putting back into service.

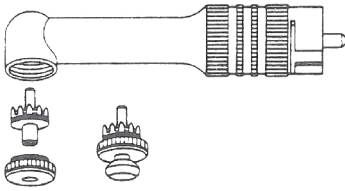Cleaning/Sterilization Instructions

1. Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
2. Autoclave angle per manufacturer's instructions. The contra angles have been tested with no adverse effects at 250° F (121° C) with a 30 minute exposure time.

Symbols

Caution: See warning or precautions

Consult instructions for use

Manufacturer

Authorized Representative in the European Union

Catalog number

Lot number

Complies with requirements of EU Directive 93/42/EEC

Rx ONLY

Integra York PA, Inc. • 589 Davies Drive, York, PA 17402 USA
866-854-8300 USA • 717-840-2763 outside USA • 717-840-9347 fax
integralife.com/integra-miltex

Miltex GmbH • Mühlstr. 6 • 78604 Rietheim-Weilheim • Germany
Phone: 49 7461/9634-14 • Fax: 49 7461/9634-30

Integra and the Integra logo are trademarks of, and Miltex is a registered trademark of, Integra LifeSciences Corporation or its subsidiaries. © 2011 Integra LifeSciences Corporation. All rights reserved.

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[EC REP] Miltex GmbH • Mühlstr. 6 • 78604 Rietheim-Weilheim • Germany