# Integra™ Miltex® INTEGRA™

## Deluxe Autoclavable Sealed Head Prophy Angles


### Directions for Use

Rx ONLY

CE 0297

REF 75-24 Screw Type REF 75-26 Snap On

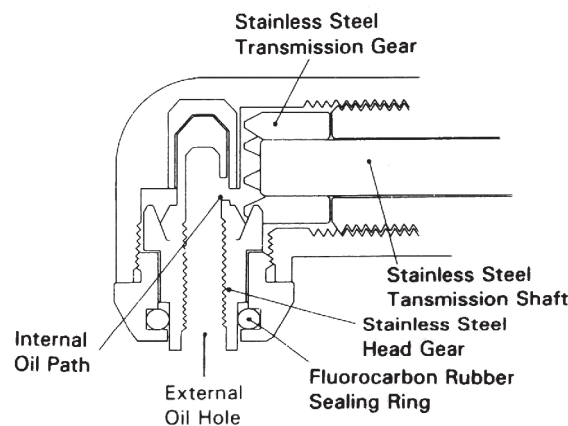#### Description

- Fully autoclavable.
- Both screw type and snap type can be oiled without disassembling the head.
- The sealing ring is made of a high quality fluorocarbon rubber and is located inside the head cap. The ring prevents prophy paste from entering the head during use.
- The gears and the shaft are made of hardened stainless steel.
- Phosphor bronze cup and head bearings.

#### Precautions

- The contra angle must be lubricated before every autoclaving procedure.
- Damage due to lack of lubrication is misuse and is not covered by the warranty.
- Maximum autoclave temperature is 270°F.
- Do not use dry heat or cold sterilization.

#### Lubrication Instructions

1. Remove cup or brush.
2. Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
3. Insert three drops of oil into the hole:

- **Screw Type** – The hole where the cup or brush is screwed in.

- **Snap Type** – The hole in the middle of the knob onto which the cup is attached.

4. Wipe off excess oil after oiling.
5. Sterilize by autoclaving before putting back into service.

#### Cleaning/Sterilization Instructions

1. Thoroughly clean the outside of the angle with a disinfectant moistened gauze sponge.
2. Autoclave angle per manufacturer's instructions. The contra angles have been tested with no adverse effects at 250° F (121° C) with a 30 minute exposure time.

#### Symbols

Caution: See warning or precautions

Consult instructions for use and website address

Manufacturer

Authorized Representative in the European Union

Catalog number

Lot number

Complies with requirements of EU Directive 93/42/EEC

Rx ONLY

Caution: Federal (USA) law restricts this device to sale by or on the order of a dentist or physician.

Integra York PA, Inc. • 589 Davies Drive, York, PA 17402 USA  
866-854-8300 USA • 717-840-2763 outside USA • 717-840-9347 fax  
integralife.com/integra-miltex

EC REP Miltex GmbH • Mühlstr. 6 • 78604 Rietheim-Weilheim • Germany  
Phone: 49 7461 /9634-14 • Fax: 49 7461 /9634-30

Integra and the Integra logo are trademarks of, and Miltex is a registered trademark of, Integra LifeSciences Corporation or its subsidiaries.  
© 2011 Integra LifeSciences Corporation. All rights reserved.

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Integra York PA, Inc. ■ 589 Davies Drive, York, PA 17402 USA
Miltex GmbH ■ Mühlstr. 6 ■ 78604 Rietheim-Weilheim ■ Germany