

MILTEX HI-LIGHT

WELDIN VETERINARY PERIODONTAL INSTRUMENT KIT FOR ROUTINE TREATMENT



The Miltex® WELDIN Periodontal Instrument Kit is a complete collection of veterinary dental instruments and accessories for oral examinations and routine periodontal prophylaxis procedures for canine and feline (companion animal) patients.

Dedicated to the advancement of veterinary dentistry since 1981, Carol Weldin, RDH, BS, in conjunction with Miltex, Inc., has specifically selected each diagnostic and working instrument and accessory for the varying length and shapes of tooth structures, accessibility to anterior and posterior teeth, sulcus depths, diversity of patients, and ease of instrument care and maintenance.

GripLite™ S[®] hygiene hand instruments feature razor-sharp, wear-resistant tips, which maintain their edges longer than most competitor tips. Each hand instrument is color-coded to match a periodontal chart to help identify where each instrument is used throughout the oral cavity. Instruments and accessories are conveniently organized in a Miltex® stainless steel sterilization cassette.

REF 6809

Kit includes 1 each of:

REF	Description
SVET-5450	GripLite™ S [®] Mirror Handle, single-ended, cone-socket
67-696/3*	#3 Cone Socket Front Surface Mirror, 1 mirror
67-696/4*	#4 Cone Socket Front Surface Mirror, 1 mirror
SVET-23E/18P	GripLite™ S [®] #23 Explorer/18mm Probe, double-ended
SVET-5004	GripLite™ S [®] H5/J33 (Jacquette H33/1YS Scaler)
SVET-767	GripLite™ S [®] Morse 0-00 Scaler
SVET-5201	GripLite™ S [®] Columbia 13/14 Universal Curette
SVET-5202	GripLite™ S [®] Barnhart 1/2 Universal Curette
DTST**	Delron Sharpening Test Sticks, Clear, 1 stick
STN-ARK-CON	Conical Arkansas Sharpening Stone, 1 stone
STN-ARK-FLT-4	#4 Flat Arkansas Sharpening Stone, 1 stone
3-084107	Miltex® Sterilization & Storage Cassette (8" L x 4-7/8" W x 1-1/8" H)

* (not sold individually – REF is 12/bx)

** (not sold individually – REF is 6/pk)

Check with distributor for individual sale.

Miltex
Redefining Excellence

www.miltex.com
toll free 866 854-8300
phone 717 840-9335

All instruments available for individual sale. For additional information, an in-office demonstration, or to place an order, please contact your local Miltex, Inc. authorized Distributor or Miltex, Inc. Customer Service at 1-866-854-8300, email customerservice@miltex.com, or visit us at www.miltex.com.






SVET-5450
67-696/3
67-696/4

1. DIAGNOSTIC - Mirror

To provide indirect vision and illumination of difficult-to-see areas. May also be used as a retractor. #3 mirror ideal for small Canines/Felines; #4 mirror ideal for large breed Canines.



#4 mirror shown

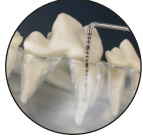

SVET-23E/18P

2a. DIAGNOSTIC – 18mm Probe (1, 2, 3, 5, 6, 7, 8, 10, 11, 12, 13, 15, 16, 17, 18 mm)


To assess periodontal pocket depths and gingival bleeding.



SVET-23E/18P

2b. DIAGNOSTIC - #23 Shepherd's Hook Explorer

To evaluate surface irregularities, caries and calculus detection, exploration of pockets, furcation involvement (in multi-rooted teeth).

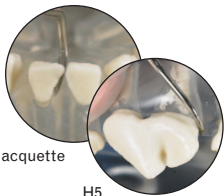

Jacquette

H5

SVET-5400

3a. WORKING – Jacquette 33 (1YS) Scaler

3b. WORKING – H5 Scaler

Designed for the removal of supragingival (above gumline) gross calculus. Ideal use on **anterior** (Jacquette) and **posterior teeth** (H5) interproximally, surface scaling, calculus deposits and stains.



33

H5





SVET-767

4. WORKING - Morse 0-00 Scaler

Designed for supragingival (above gumline) scaling of crowded and/or overlapping teeth. Excellent for stain removal.

- Ideal for stain removal of the carnassial teeth #108 and #208 (upper arch, 4th pre-molars) in Canines.
- Ideal for stain removal of the carnassial teeth #104 and #204 (upper arch, canine teeth) in Felines.

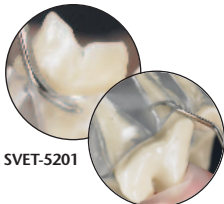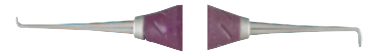

SVET-5201

SVET-5202

5. WORKING – Columbia 13/14 Universal Curette

6. WORKING – Barnhart 1/2 Curette

Columbia 13/14 Universal Curette is designed for removal of subgingival (below gumline) and supragingival (above gumline) calculus on **buccal and lingual surfaces of posterior teeth**. Ideal for Felines and small/medium breed Canines.

Barnhart Curette is deal for larger Feline teeth and large breed Canines.




Columbia Universal Curette



Barnhart Curette

Sterilization & Storage Cassette

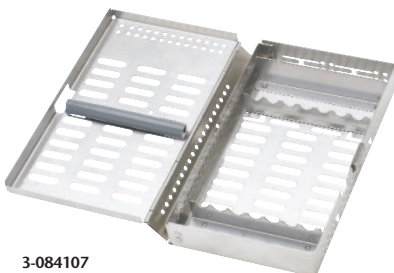

3-084107

Acrylic Test Stick

Used to test the sharpness of hand instruments.



DTST

Periodontal Procedure Chart

6809C



#4 Flat Arkansas Stone

Used to sharpen hand instruments.



STN-ARK-FLT-4

Conical Arkansas Stone

Used to complete the sharpening process in removing metal debris from hand instruments.



STN-ARK-CON