MILTEX CONTAINERS

Miltex Containers are designed to provide a more efficient organization and utilization of surgical instruments for the operating room, surgi-center and central supply. The functional containers are manufactured from an anodized aircraft aluminum alloy to prevent corrosion. This aluminum alloy also has high thermal conductivity for faster drying and is lighter than other metals to make the Miltex Container easier to transport.

PRE-CLEANING

- Remove lid and basket with instruments from the Container.
- Remove the lid filter retention plate(s).
- If using perforated bottoms, remove the bottom filter retention plate(s).
- Discard the disposable filter(s).
- Remove all indicators and disposable locks.

CLEANING

Do not use any abrasive products, material or stainless steel brushes to clean the Miltex Containers. Abrasives can damage the aluminum surface of the container and will void all manufacturer warranties. **Never expose to bleach or other corrosive chemicals.** Exposure to bleach may result in pitting or damage to surface and will void all manufacturer warranties.

Manual Cleaning

- Use a soft sponge and a neutral pH (7) phosphate-free detergent such as Miltex Surgical Instrument Cleaner (3-720, 3-725 and 3-726) to thoroughly clean the Miltex Container. Use distilled (demineralized) water (room temperature) if possible.
- Rinse with water to remove the cleaning solution. Be sure to rinse thoroughly to remove all detergent residue.
- Dry thoroughly. Store in clean and dry environment.

Mechanical Cleaning

- Place the container bottom face down (open side down) in the washer.
- Handles should be folded down.
- Dry thoroughly. Store in clean and dry environment.

ASSEMBLY FOR USE

Pre-Assembly Inspection and Preparation – All components must be thoroughly dry before assembling

- Select surgical instrument set for sterilization.
- Select appropriate size container and basket(s).
- Inspect the inside channel of the lid to ensure the silicone gasket is in good condition and free from cracks. A cracked gasket could indicate age and/or deterioration and should not be used.

Filter Assembly

- Place one sheet of filter paper over each perforated section on the inside of the container lid or bottom – making sure it is centered on the protruding pin. When using filter paper with indicator, be sure the indicator faces toward the inside of the container when the lid is in place and the indicator is visible through the retention plate.
- Secure each filter with the retention plate by pressing onto the protruding pin until you hear it click.
- Perforated bottoms: Place one sheet of the filter paper over the perforated section in the bottom of container – making sure it is centered on the protruding pin. Secure the filter with the retention plate by pressing onto the protruding pin until you hear it click.
- Solid bottoms: There is no perforated section and does not require a filter or retention plate.

Instrument and Container Assembly

- Place cleaned and dried instruments into the instrument basket(s) according to established hospital procedures. Complex instruments (e.g. air-powered instruments, endoscopes, and instruments with lumens or channels) should be prepared and sterilized according to the instrument manufacturer's instructions. The containers have been validated for sterilization of instruments with lumens up to 3 mm I.D. by 400 mm length, for the full (large) size container and up to 3 mm I.D. by 200 mm length for the 1/2 (small) and 3/4 (medium) size containers. The containers are intended for use with stainless steel surgical instruments, that is, non-porous materials.
- Small baskets, trays, and other types of accessories, especially with covers and lids, should only be used with the sterilization container if the container has been specifically designed and tested for that purpose. Non-absorbent tray liners (e.g., plastic/silicone-fingered organizing mats) can cause condensate to pool.
- Place instrument basket(s) into the prepared container bottom.
- Place lid on to the container bottom, aligning handles on bottom with latches on lid.
- Simultaneously close both locking latches on the lid. Some downward pressure is needed to lock the lid in place.

NOTE: For effective sterilization and drying, the weight of the basket and basket contents should not exceed the following:

- 16 lbs for 1/2 (small) size containers
- 20 lbs for 3/4 (medium) size containers
- 25 lbs for Full (large) size containers

All instruments should be assembled to allow for uniform exposure to sterilization agents.

Caution – Leave minimum of one inch clearance for effective processing

Processing Assembly

- Insert indicator card(s) into the holding bracket(s) on the outside of the container. *A tab at one end of the indicator card will facilitate insertion and removal.*
- Insert the plastic tamper-proof seals into the locking channel on each end.
- Secure and lock.

NOTE: Use of internal and external indicators should be in accordance with in-house protocol, determined by the user.

Loading the Sterilizer

- The container should be placed **flat/level** for effective sterilization and drying.
- The container should be positioned on the autoclave cart below wrapped items for optimum sterilization and drying conditions.
- **Stacking:** Only containers with perforated bottoms should be stacked; do not stack containers with solid bottoms! Stacking should not exceed 18" in height for effective air removal and adequate steam penetration.

CAUTION: Do not cover filter perforated section with paper wrapped instruments or any other objects that could obstruct the way to air flow and steam flow. Permanent damage may occur to the container as a result of this obstruction.

PROCESSING

- Run loaded sterilizer according to the manufacturer's instructions at:
 - High Vacuum (pre-vacuum, three pulse, standard)
 - Temperature: 270° F
 - Exposure Time: 4 Minutes (minimum)
 - Cycle Dry Time: 20 Minutes (minimum)
 - Cool Time: Varies according to load contents

CAUTION: Cool drafts from air ducts or other air currents should be avoided during the cooling phase to avoid post-sterilization moisture caused by rapid cooling syndrome.

- The container needs to remain on the container cart, in a draft free area, until cool enough to handle.

NOTE: A Container with a solid bottom may require additional cooling time; the additional required cooling time to be determined by the user.

THE MILTEX CONTAINER SYSTEM WARRANTY

Miltex hereby warrants all Miltex Containers purchased directly from Miltex or an Authorized Dealer in accordance with the following terms and conditions:

The Miltex Container is guaranteed to be free from defects in material and workmanship when used for its intended purpose. Any Miltex Container that proves defective in workmanship or material will either be repaired or replaced, at Miltex discretion, without charge.

Containers not purchased from an authorized source may not be authentic or meet Miltex quality standards.

Container Gaskets Warranty

The warranty period for Miltex container gaskets is three years from the date of purchase.

Periodic Maintenance

Periodic Maintenance is suggested to ensure all parts are in good working order.

Repair or Replacement

Under this Warranty, and at its discretion, Miltex will repair or replace any parts of the product found to be defective in workmanship or material.

Miltex will determine, at its discretion, the cause and nature of any defect, the necessity and manner of repair or replacement, and all other matters pertaining to the conditions of the Miltex Container System.

Miltex, Inc. Returned Goods Policy applies to all returned items (refer to details in current Surgical price list).

Exclusions

This Warranty shall not apply to any conditions(s) or damage resulting from negligence, misuse, improper handling, improper cleaning, improper opening techniques, or unauthorized repair work including but not limited to:

- Incidents of abuse such as denting of the Miltex Container due to dropping or other instances of mechanically applied pressure.
- Damage determined to be related to caustic or abrasive cleaning agents.
- Items modified by the customer.
- Items modified or customized by Miltex at the request of the customer.
- Damage from fire, flood and other occurrences not under the control of Miltex, Inc.

Please direct your questions concerning this Warranty to: Miltex, Inc., 589 Davies Dr., York, PA 17402

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- Schäden durch Feuer, Überschwemmungen und andere Vorkommnisse, die nicht der Kontrolle von Miltex Inc. unterliegen.
Bitte richten Sie Ihre Fragen bezüglich dieser Garantie an: Miltex, Inc., 589 Davies Dr., York, PA 17402